










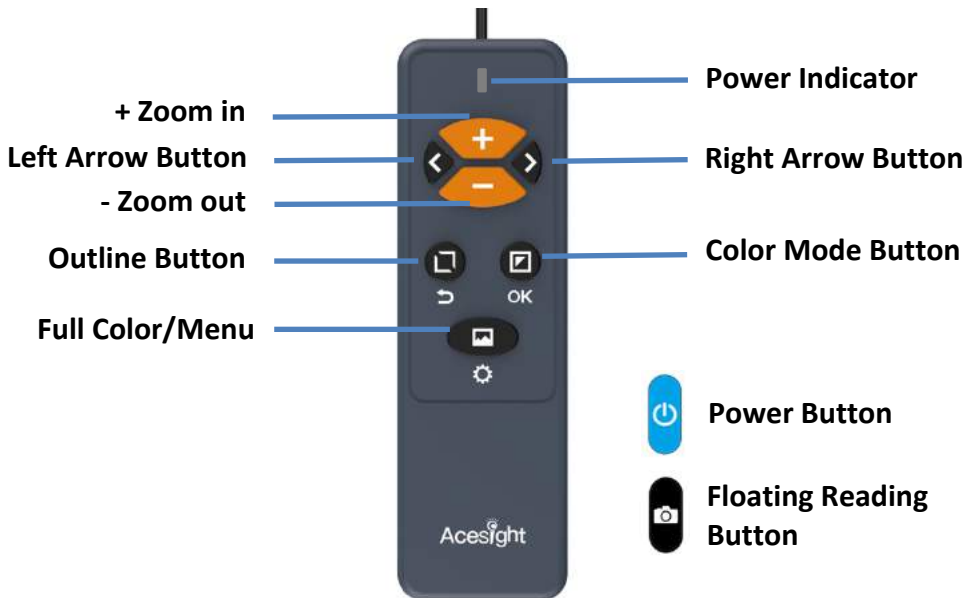



Quick Guide






V 1.0





USB Port for Firmware Upgrade




Basic Function	Instruction
Turn On/ Off	Press Power Button  for 2 seconds.
Zoom In	Press Zoom In Button  .
Zoom Out	Press Zoom Out Button  .
Color Mode	Press Color Mode Button  to circle through color modes.
Outline	Press Outline Button  .
Adjust Thickness of Outlines/Adjust Contrast of High Contrast Colors	Press Arrow Buttons   .
Switch to Full Color Mode	Press Full Color/Menu Button  .
Lock/unlock the Controller	Press the Zoom In and Zoom Out Button   simultaneously.

Advanced Function		Instruction
Floating Reading Mode ¹	Step 1: Enter Preview	Short press the Floating Reading Button  . A yellow square with a + symbol will appear  .
	Step 2: Enter Floating Reading Mode	Press the Floating Reading Button  again to freeze the image you want to see.
	Step 3: See with Floating Reading Mode & Exit	Move your head around to view the frozen image. If image disappears, press Full Color/Menu button  . To exit, press the Floating Reading Button  .





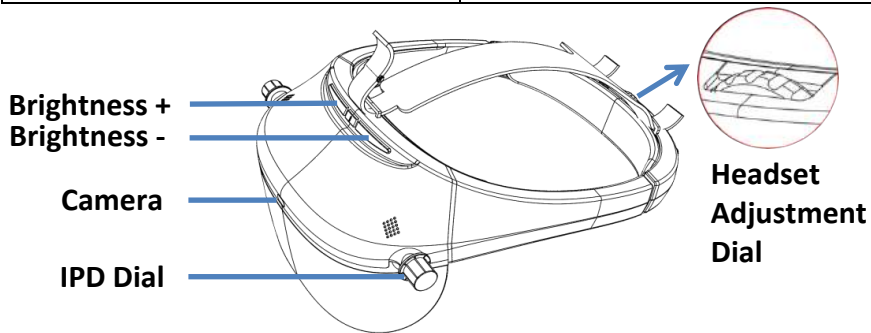


¹ Floating Reading Mode makes your eyes less tiring when reading longer documents. For more information, please contact your dealer/doctor.

Advanced Function	Instruction
Lock/Release Focus ²	Press Arrow Buttons   simultaneously.
Adjust the Interpupillary Distance (IPD) ³	Close the left eye and adjust the IPD dial on headset for right eye. Repeat for the other side.




² Please lock the focus when reading texts, and unlock it when looking around.

³ Please contact your dealer/doctor for assistance if there is any problem adjusting the IPD.

Advanced Function		Instruction
Access the Menu	Enter/Exit Menu	Press Full Color/Menu Button  for 3 seconds to enter Menu. Short press it again to exit.
	Enter/Exit Options	Press Mode Button  to enter select options. Press Outline Button  to exit.
	Switch Options	Press Arrow Buttons   to switch between Floating Reading Mode Calibration ⁴  and Factory Reset  .

⁴ It's not suggested to use Floating Reading Mode Calibration Function without professional advice.

Specifications

Display	Full HD display, 1920*1080 resolution
Camera	8 megapixels
Field of view	45° (per eye)
Refresh rate	60 Hz
Magnification	1.1x-15x
Display modes	Full color and 4 high contrast color modes 
Usage time	4+ hours
Weight	Headset: 410g (0.8lb) Controller: 183g (0.4lb)

Zoomax Technology Co., Limited

Tel: +86 (0571) 8700-6308

Email: support@acesight.com

Visit us at www.acesight.com